

Blood sample Collection and Shipping Procedures

Sample Collection:

1. Blood should be collected in a purple-top EDTA tube.
 2. Collect 3-5 ml (cc) per individual.
 3. Label the tube(s) with the animal's name.
 4. Send the sample ID form as well as other relevant forms along with the tube.
-

Shipping:

1. It is critical that the samples be shipped immediately using FedEx or UPS OVERNIGHT (next business morning) shipping.
2. Samples must be shipped in an insulated box with an ice pack. (Check with Miriam Aguilar – in some cases you can put the sample in a small sturdy box – wrap blood tubes in paper towels and put them in a Ziploc bag, and then wrap with bubble wrap or lots of padded paper. **ICE PACK IS REQUIRED**.)
3. Address to:

**Bannasch Lab c/o Miriam Aguilar
UC Davis School of Veterinary Medicine
Dept. of Population Health and Reproduction
4206 VM3A
One Shields Ave
Davis, CA 95616**

If you have any questions please contact Miriam Aguilar
phone: 530-754-7289 email: miraguilar@ucdavis.edu